


America's #1 Selling Dental Articulation Tray System

Quadrant Tray

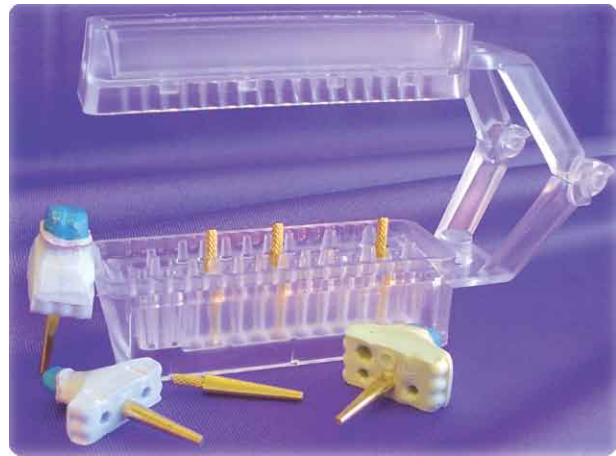

***Follow the proven success!
Reduce remakes and dentists chairtime
while saving on materials & labor***

Benefits

- 3 different types of trays customized to work with your die stone linear expansion rate
- Zero expansion premium polymer base: Far less interproximal contact & high occlusion problems
- Superior Accuracy & Perfect Die Seating
- Fast, Simple & Easy... Efficient & Versatile
- Patented pinhole covering methods


U.S. Patent Nos.: 7,147,465, 6,948,932 and Additional Patents Pending

Create Quality by
Reducing Variables & Increasing Accuracy...
eliminate uncontrollable and unaccounted for base stone expansion



www.ArtimaxUSA.com

3 types of trays... Customized for YOUR Preference

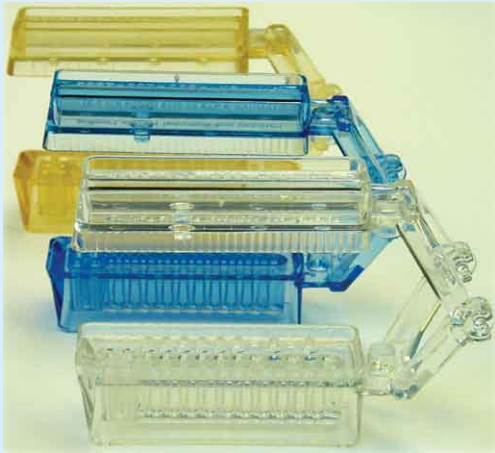


Choose your version based on...

- your die stone expansion rate
- ceramists' preference
- model technician's preference
- whether you have contact problems

Choose whichever type of stone best suits your lab and Artimax will do the rest. With 3 choices of trays you are sure to find complete compatibility.

Articulator Tray	Die Stone Expansion			Main benefits
	Low	Med	High	
Wall type	●	●		Maximum retention & die stability Comparatively easier for beginners
Wall-less 1	●	●	●	Easier confirmation of complete die seating Recommended for high expansion stones
Wall-less 2	●	●	●	Recommended for labs experiencing contact problems consistently

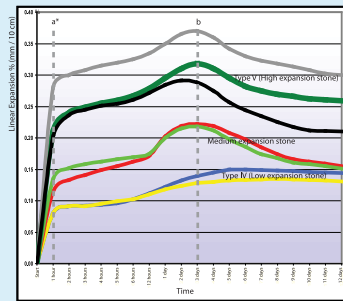

* All 3 types of articulators are currently available in clear and transparent blue

Technique Guide

Takes only 5 minutes per case. Everything you need is already built into the Artimax articulator



Pour the opposing side first. This will help stabilize the impression position on the tray.



Mark the center of the prepped tooth on the impression and then on the working tray.



Insert the pin into the marked pinholes, puncturing the membrane that covers the pinholes



Pour the die stone onto the working tray and invert over the impression. Let the stone set.



Finished model in 5 minutes.

- 3012 Quadrant manual (1 page - simplified)
- 3018 Quadrant manual (8 pages - detailed, step-by-step)



Download the manual

Ordering Information

- 2010 S Quadrant Trial Kit (Small)**
20 Quad trays (any combination of version & color)
- 2010 M Quadrant Trial Kit (Medium)**
40 Quad trays (any combination of version & color)
- 2010 L Quadrant Trial Kit (Large)**
50 Quad trays (any combination of version & color)
- 1010 Quadrant Wall version (Clear 1010 C/ Blue 1010 B)**
50 Wall version quad trays, 200 medium size brass dowel pins, 10 posterior stop rods
- 1020 Quadrant Wall-less version 1 (Clear 1020 C/ Blue 1020 B)**
50 Wall-less version quad trays, 200 medium size brass dowel pins, 10 posterior stop rods
- 1030 Quadrant Wall-less version 2 (Clear 1030 C/ Blue 1030 B)**
50 Wall-less version quad trays, 200 medium size brass dowel pins, 10 posterior stop rods



Our Guarantee - If you are not 100 % satisfied, simply return the product for a complete refund. It's that easy.